


# BYRNE™

## BYRNE FACE SHIELDS KIT

Includes ten (10) Face Shield packages per order.



### PACKAGE INCLUDES

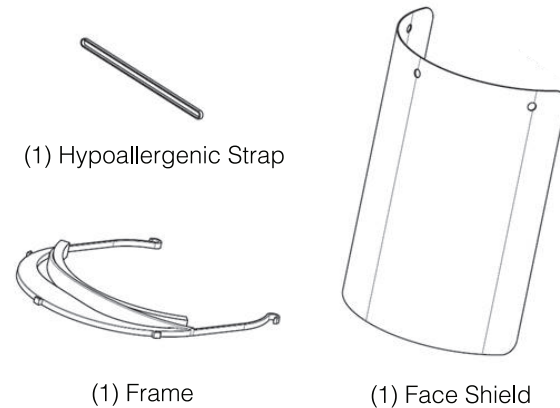

### ASSEMBLING YOUR FACE SHIELD

- 1 Remove protective film from both sides of the face shield.
- 2 Attach the face shield to the frame by placing the holes on top of the three mounting points on the frame.
- 3 Rotate the frame counter-clockwise or the face shield clockwise until it is in place.
- 4 Attach the hypoallergenic strap to the rear-side of the frame.
- 5 Place the rear of the frame on the back of your head and slide the frame over your head until the front of the frame is on your forehead and the face shield covers your face.

### DISCLAIMER

Disclaimer: for professional use only. Byrne Electrical Specialists, Inc., their affiliates, employees, officers directors and agents ("Byrne") makes no representations, guarantees or warranties of any kind with respect to the part(s) ("product") or the production and/or use of this product, whether express or implied, including any guarantee, representation or warranty relating to (1) the compliance of the equipment or facility to manufacture product that may be covered by regulation, (2) the compliance of any product with any specific regulation, (3) their condition, (4) conformity to any description, or (5) the existence of any latent or patent defects in the design, the materials or product; or any warranty of merchantability, fitness for a particular purpose or non-infringement. Customer assumes the entire risk relating to the use or performance of the equipment, the design, the materials or product and Byrne shall not be liable for any defect, deficiency or nonconformity in the design, the materials or product. Customer is solely responsible for the evaluation of and determination of the suitability and compliance with applicable regulations of the equipment, designs and/or

product for any use, especially for uses (including but not limited to medical, dental, food contact, automotive, aerospace, heavy industry, and consumer products) that are regulated by the United States, European Union and/or other applicable regulatory agencies or governments. Given the exigent circumstances under which this product is being made, Byrne makes no representation as to the safety and efficacy of this product. You understand this product is meant to benefit society, and is not intended, and shall not be used, for commercial use. This product has not been cleared for medical use by the U.S. Food and Drug Administration nor is it intended to replace surgical masks and shields, and has not been evaluated for safety or effectiveness for any medical condition. We are making these materials available as emergency responses to the shortage of personal protective equipment. If you have questions or concerns regarding this product, associated designs and/or information provided on this website, please contact us.